# Post-Op-Schuhe

by **schein**

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Vi ønsker dig en god helbredelse.

## Instructions for use

### For patients and orthopaedic specialists

Dear Customers,

Thank you for the trust you have placed in us. You have made a good choice with your purchase. We set ourselves the aim and standard of ensuring the best possible care of the foot with our shoes, in close cooperation with our partners, doctors, health insurance companies and medical services. To find out more about our products visit our website at:

[www.schein.de](http://www.schein.de)

Before using our **medical device**, please read the instructions for use carefully and pay attention to the following advice, in particular the safety information:

#### 1. Purpose/indication:

Surgical shoes:

- Impairment of mobility (walking) in the event of damage to the skin and subcutaneous tissue in the course of conservative or post-operative treatment of wounds and ulcers lasting a few weeks or longer with extensive, possibly padded dressings, the healing process of which allows at least partial load on the foot when walking. Suitable in the case of post-traumatic care following forefoot fractures or bruises.
- In case of strong tendency of the feet to swell, e.g. as a result of heart failure, lymphoedema or post-thrombotic syndrome, when conventional ready-to-wear shoes no longer fit.
- The post-op shoes are intended to enable and maintain appropriate capacity for walking.

Forefoot and hindfoot relief shoes:

- Moderate to severe damage to the foot during conservative or post-operative treatment (= early mobilisation).
- With diabetic foot ulcers, only if no full-contact plaster or comparable orthosis is possible.
- Post-traumatic care after forefoot/hindfoot fractures or bruises.
- With open wounds and ulcers, especially on the forefoot/heel, e.g. as a result of arterial or venous circulatory disorders or injuries.
- In case of painful swelling of the forefoot, e.g. due to an acute attack of gout or acute bursitis.
- In case of painful, inflammatory diseases of the hindfoot, e.g. acute heel spur symptoms or acute bursitis at the base of the Achilles tendon.

#### 2. Location/conditions of use:

- Make sure that you only move on firm surfaces.
- Protect your foot and, if relevant, your dressing from dirt, impacts and moisture if you use the post-op shoe outdoors. You can use an insert sheet for protection if necessary. Discuss this with your orthopaedic (shoe) specialist retailer if required.

**Please note that there may be a risk of slipping on moist or smooth ground, as well as snow and ice. Special care must be taken under such circumstances.**

#### 3. Application instructions:

- The dimensions of the shoe have been changed in the affected area for the purpose of your treatment. Take this into account, especially when climbing stairs or stepping over obstacles.
- Be aware that you should wear a firm shoe with appropriate height adjustment on your healthy foot.
- Undo and loosen the fasteners before putting on the shoe. Put on the shoe and fasten them.
- These are medical devices that are only ever intended for a single patient. Use by other persons is not permitted.
- Always sit to put on your shoes.
- The wearing period and frequency of use will be determined by your doctor.

Note: In the case of Malum perforans as a result of peripheral neuropathy or in arterial peripheral circulatory disorders, the load-bearing capacity of the rest of the foot is restricted, meaning that in such cases additional relief through a lower-leg orthosis is recommended.

**Please take note of our instructions before each use of the post-op shoes, and especially before the first use**

#### 4. Safety tips and risks of use:

- Feel the inside of the shoes before each use and remove any foreign objects.
- Never use the post-op shoes without a suitable walking aid.
- Do not drive a vehicle yourself.
- Only wear the post-op shoes if there is no direct skin contact. Wear them either with a dressing or with appropriate protection for your foot and/or wound.
- If there are pressure points, pain or other negative sensations, consult your doctor or the service provider immediately.
- To avoid pressure points and circulatory disorders, do not fasten the shoe too tightly. Check the fastening and adjust it if necessary. If the fastening is too loose, this leads to shifting within the shoe, which must also be avoided.
- Shoes fastened too loosely can lead to falls or pressure points due to shifting.
- You should check your feet daily for redness and pressure points.
- Due to the soft shaft material, the shoe has less support against shear force than solid leather shoes, meaning that ankle twisting on pavement or dirt tracks is possible.
- The dressing can lead to incorrect sizing; in the worst case this can lead to pressure points.
- Incorrect usage could put strain on the area where it is supposed to be reduced.

#### Important note:

**In case of problems with your feet or shoes, please contact your doctor or service provider immediately.**

## 5. Contraindications:

- If you suffer from existing skin allergies to the materials used, please consult your doctor before use.
- Use the shoes only if partial load is manageable.
- In the case of significant foot deformities, a ready-to-wear post-op shoe is not suitable.

## 6. Material characteristics:

- **Upper material:** Nolen with polyurethane padding layers/textiles
- **Lining material:** Velor
- **Footbed:** Ethylene vinyl acetate (Multiform)
- **Outer sole:** Ethylene vinyl acetate

The materials for the lining and footbed are the same for all models.

## 7. Cleaning and care instructions:

### Before the first use

- The attached Velcro fasteners are kept very long, so that they can be adapted and shortened individually to each patient. Therefore, please have the Velcro fasteners shortened by specialist personnel.

### Cleaning

- When dirty, the post-op shoes can be machine-washed at 30°C on a gentle cycle. Leave the shoes to air-dry.

### Disinfection

- If necessary, you can clean the lining material with a damp cloth.
- We recommend a disinfectant that may be used with open wounds to clean the lining. Seek advice from your doctor or pharmacist on this point.

### Drying

- Never put wet or damp shoes near a direct heat source to dry.
- Please avoid using radiators and direct sunlight to dry the shoes.
- Give your shoe time to dry. This can take up to two days, depending on the moisture penetration.

### Storage

- Please store shoes dry and at room temperature.

## 8. Disposal:

- Disposal of the product and packaging must be carried out in accordance with statutory provisions.

## 9. General information / maintenance:

- The post-op shoes are a medical device. Your doctor determines the wearing period, which must be adhered to.
- We recommend wearing the partial-relief shoes no longer than three months and the surgical shoes no longer than six months.
- Check the shoes regularly for functionality, wear and tear or damage to the fasteners, soles, etc.
- In case of loss of function, wear and tear or damage, you should no longer wear the shoes and should consult your doctor or orthopaedic (shoe) technician. The shoe must be replaced by a new one.
- We also recommend that you have a regular, suitable

check of your foot and your wound carried out.

## 10. Assembly instructions (specialist personnel):

- Remove the shoe fillers (e.g. paper) and adjust the shoes.
- Shorten the Velcro straps to the appropriate individual length for your customer.
- Insert the enclosed padded sole correctly into the shoe.
- In the case of individual changes to the product, please note that the medical aid must be permanently marked with the label 'Custom-made product'.

## 11. Warranty:

- We grant the statutory warranty for use as intended.

Please inform the company schein orthopädie service KG in case of anomalies, problems or defects with the shoes. We wish you a speedy recovery.

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